




## Spirobank II Basic Quick Start

1. The Spirobank II Basic can be turned **on** or **off** by holding **this button**.

There will be 3 screens that appear:

The first is the MIR logo splash screen.

This shows the date & time

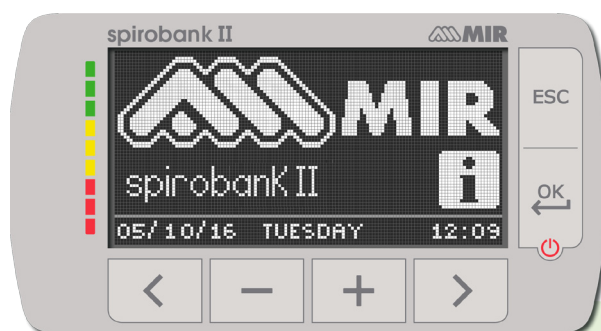

### MIR USA, Inc.

5462 S Westridge Dr., New Berlin, WI 53151

P +1 (844) 464 - 7872 F +1 (262) 364 - 2030

pousa@spirometry.com www.usa.spirometry.com

### MIR ITALY, Head Office

Via del Maggiolino, 125 - 00155 Roma

P +39 06 22 754 777


F +39 06 22 754 785

mir@spirometry.com

www.spirometry.com

**The second screen shows**

the number of patient files,  
the number of Spirometry tests,  
the amount of free memory  
and the battery symbol  
indicating amount of battery charge.

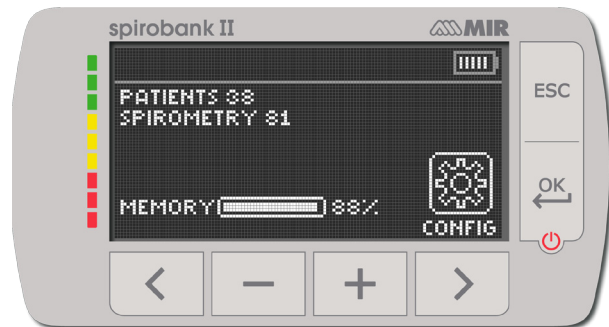

It will also show you the  
configuration icon that brings you to the  
**Configuration Menu.**

Pressing the Right Arrow  
under this symbol will take you to the  
**Configuration Menu.**

**The third screen is the Main Screen  
where you can choose a function**

You can also navigate the  
configuration menu from the main screen  
pressing the **ESC** button



Use the left & right arrow buttons to scroll up  
and down through configuration menu

## Entering patient information

Press the left arrow  
under the I.D. symbol



This screen appears:



## There are 2 choices for entering patient information


**Choose New** by pressing the left arrow  
to create a new patient.

**Choose Modify** by pressing the right arrow  
to edit an existing patient file.

## Selecting New patient: the insert patient data screen will appear.

Use the minus and plus buttons to increase/decrease  
numbers in the number fields.

Once birthdate, height (inches or cm), and weight  
(lb or kg) have been entered 2 choices will appear  
for gender press the minus to highlight male or  
press the plus sign to highlight female.

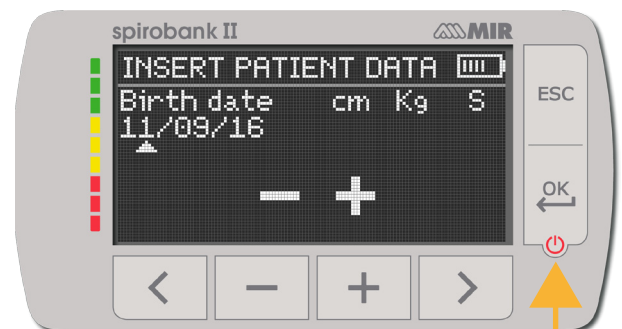

Then press the **ok button** to make your choice  
Right after that the ethnicity options will appear  
Use the left and right arrow to scroll up and down through the choices  
press the ok button to confirm your choice.

**Now you will be back on the main screen**

### MIR USA, Inc.

5462 S Westridge Dr., New Berlin, WI 53151  
P +1 (844) 464 - 7872 F +1 (262) 364 - 2030  
pousa@spirometry.com www.usa.spirometry.com

### MIR ITALY, Head Office

Via del Maggiolino, 125 - 00155 Roma  
P +39 06 22 754 777 F +39 06 22 754 785  
mir@spirometry.com www.spirometry.com

## To Perform an FVC test

Insert the FlowMIR turbine with the cardboard mouthpiece pointing out the same side that screen is on and twist it clockwise to lock

Back on the main screen.


Press the minus sign under the spiro symbol



**A new screen appears with spirometry test choices.**

(The POST choice will be grayed out until at least one Pre FVC test is done).

Press the button under the choice of test.

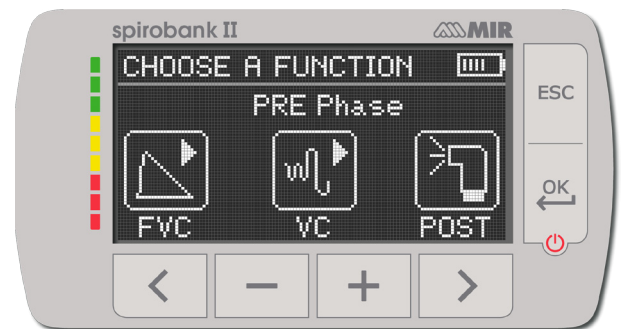

The next screen will have instructions about performing the test.

The patient may begin to perform the test at his screen.

The test will time out if not started in time

## Connect the device to the computer via the micro USB cable to charge it.

The screen will say 'external control' and will remain on while connected.



### MIR USA, Inc.

5462 S Westridge Dr., New Berlin, WI 53151

P +1 (844) 464 - 7872 F +1 (262) 364 - 2030


















pousa@spirometry.com www.usa.spirometry.com

### MIR ITALY, Head Office

Via del Maggiolino, 125 - 00155 Roma

P +39 06 22 754 777 F +39 06 22 754 785

mir@spirometry.com www.spirometry.com

-  To access the default configuration menu
-  To access patient data from the main display
-  To perform a new test of a patient recalled from the patient records
-  To insert new patient data
-  To modify patient data
-  To display the most recent test of a patient
-  To show the last test performed
-  To access the database of the performed tests
-  To search a test with the date of birth of a patient
-  To search a test starting from a specific date onwards (partial database)
-  To flick through a database from beginning to end and viceversa (complete database)
-  Male patient selection
-  Female patient selection
-  To access spirometry testing type
-  To perform a forced vital capacity spirometry test FVC/search FVC tests in memory
-  To perform a slow vital capacity spirometry test VC/search VC tests in memory
-  To perform a spirometry test with a broncodilator